uTV 2 Technical data

Difficulty hearing the television is a common problem for those with hearing loss, and can be a source of irritation for their loved ones when the TV volume must be turned up uncomfortably high. Now there's a solution that makes hearing the TV enjoyable for everyone again. Together with uDirect[™] 2, uTV[™] 2 delivers exceptionally high-quality stereo sound directly to the hearing instruments, to meet the listening needs of those with hearing loss along with everyone else in the family.

Description

Buttons

- (a) ON/OFF button
- (b) TV signal volume (up)
- (c) TV signal volume (down)

Inputs

- (d) Power input (mini-USB)
- (e) Audio input (3.5 mm jack)
- (f) uDirect 2 charging slot

Status indicator

(g) Status light

Special features


Compatible with uDirect 2 Audio signal connectivity test Missing audio input visualization Easy and quick installation Complete set with power plug and cables No line of sight to hearing aids necessary Up to 30 meters (100 ft) transmission range Digital volume adjustment with audio-visual feedback Integrated charging slot for uDirect 2


General information

Dimensions:	L x W x H: 90 x 60 x 23 mm (3.8 x 2.5 x 1 inches)	
Weight:	100 g	
Color:	Light metallic silver	
Operating conditions:	The product has been designed for trouble-free operation without restrictions when used as intended, unless otherwise stated in the user guide -10°/55° C and relative humidity of <95% (non condensing)	

Storage and transportation conditions:

-20°/60° C and relative humidity of 90% for a long period of time

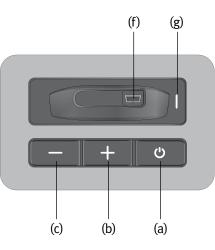

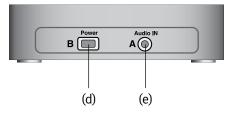



Status light indication

---- Power ON with audio input

- - Power ON, no audio input

- Transmitting to uDirect 2
- Paused transmission
- •••• Bluetooth pairing

Manual Bluetooth pairing: Press ON/OFF button for 5 seconds

Delete Bluetooth pairing table: Press ON/OFF button for 10 seconds

Audio test mode*

Press ON/OFF button and volume (-) button until test melody plays in the hearing aids and status light blinks. *uTV 2 only

Power supply

Туре:	DC, stabilized
Secondary voltage:	5 VDC
Max. current:	500 mA
Plug:	Mini-USB
Important notice:	Only use the original Unitron power plug or a USB certified power adaptor > 500 mA

Included accessories

Audio cable 3.5 mm plug Audio cable RCA/Cinch SCART adaptor (optional) Power plug (country specific) User guide Quick setup guide

Audio information

Audio input:Wired (3.5 mm socket)Audio quality:Stereo, 20 Hz - 22 kHz

Based on Bluetooth technology

Bluetooth [®] profile:	A2DP, Advanced Audio Distribution Profile
	Optimized for use with uDirect 2
	Compliance to non-Unitron devices is not guaranteed
Frequency range:	2402.0 to 2480.0 MHz
RF power:	100 mW max.
Bluetooth range:	Up to 30 m (100 ft)
Bluetooth compliance:	Bluetooth version 2.1, Class 1
Maximum paired devices:	4
QDID:	B018562

Compatibility note

Make sure the following combination is used:



uTV 2

Round shape



uDirect uTV Rectangle shape

Standards applied

Streamer:

Charging slot:

Characteristics:

Europe	
Radio:	EN 300 328
EMC:	EN 301.489
El. Safety:	EN 60950-1
USA	CFR 47, part 15
Canada	RSS-210 Issue 8

Bluetooth[®]

The Bluetooth[®] word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Unitron is under license. Other trademarks and trade names are those of their respective owners.



This device is a class 2 device and can be operated in the EU member states without restrictions for indoor use, but must not operated outdoors in France.

